RUS

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ENG

MAIN INFORMATION

LD-N057 stabilized power source is designed for transforming of 200V AC50/60 Hz voltage into stabilized (DC) 6V voltage with maximum rated load of 600 mA. Recommended as power source for LD and NISSEI brand automatic blood-pressure and pulse meters.

Constructively the adapter consists of the monolith plastic case with rigidly attached power plug. The internal components are fixed on the plated circuit. There is a green power indicator on the case.

GENERAL INSTRUCTIONS

If the adapter was stored under 10°C keep it in warm place and start operating it not less than after 12 hours. Before use, read these OPERATING INSTRUCTIONS.

HOW TO USE

1. Check state and consistency of cords, insulation and the case.
2. Firmly install the power plug of the adapter into the DC input socket of the device you are going to use
3. Connect the power plug to AC wall outlet.
4. The adapter is ready for use.

SPECIFICATIONS

Model	LD-N057
Output voltage, V	6 ±5%
Max. output current, mA	600
Input voltage, V	200-240, 50 Hz
Rated power consumption, mA	35
Dimensions, mm	81(L) x 50(W) x 71(H)
Net weight, kg	not more than 0.3
Connecting cord length, mm	1800 ± 10
Power plug:	
Output plug polarity	“-” inner
Input diameter, mm	2.1 ± 0.1
Outer diameter, mm	5.5 ± 0.1
Length of the plug's contact part, mm	10 ± 0.3

Operating conditions:

Operating temperature	between +10 °С and +40 °С
Humidity	up to 85 %Rh under 25 °С
Barometric pressure	650 - 800 mmHg

Storage and transportation conditions:

Storage and transportation temperature	between -20 °С and +50 °С
Humidity	up to 85 %Rh under 25 °С
Barometric pressure	650 - 800 mmHg

The set includes LD-N057 AC adapter, operating instructions, warranty card, package.

Operating life 7 years

SAFETY INSTRUCTIONS, STORAGE, REPAIRS AND UTILIZATION

Before disconnecting the AC adapter and the device it is used for, make sure that the AC cord is removed from the wall outlet. Do not use the device without attention. Don't disassemble the AC adapter. Should the damage occur contact authorized repair shops only. Prevent the unit from contact with water, chemical solvents, alcohol and benzine. Don't keep the AC adapter close to sources of heat or open fire. Prevent it from dirt and dust. The AC adapter is intended for indoor use only. After the nominal life period expires it is necessary to check its state and technical condition periodically. Consult to authorized service personnel (authorized repair shops). When utilizing the AC adapter follow the current regulations, adopted by local authorities. The maker does not establish any rules for utilization of the unit.

GUARANTEE

If the unit should become defective during the guarantee period as result of the makers' workmanship this guarantee provides the Customer a right to free of charge elimination of this defect. The unit is covered be guarantee for a period of 12 months from the date of purchase. This guarantee does not cover the goods which broke down and/or became defective because of use of the unit beyond normal operating conditions, inaccurate use which caused damage, any attempts by an unauthorized organization (or person) to modify, disassemble the unit and/or repair it, neglect of normal operating rules and/or prescribed storage and/or transportation conditions, actions of a third party or force majeure.

In case any questions concerning the use of the unit arise, please consult to the organization, authorized to offer guarantee services. In case there is no such organization at your area, contact the sales outlet where the unit was purchased.

The Customer can get addresses of the organizations authorized to provide guarantee services from authorized importers of the unit or from Internet web-site WWW.LITTLEDOCTOR.SG. To obtain guarantee services, the Customer should produce the guarantee coupon, the document, confirming purchase of the unit (receipt of the cashier's terminal) and the unit itself. The guarantee coupons which are not filled in or filled in only partially, which have erased data or corrections, are not accepted.

The organizations authorized to provide guarantee services receives the unit for inspections of its condition and guarantee covered repair. Under the law in force guaranty remedy should be done within 45 days.

The Customer's claims and advices can be sent at the authorized importers' addresses:

Russia: 117218, Moscow, PB 36, « K & K» Ltd. (Registration address: 105484, Moscow, 16-Parkovaya str. 35A) Tel.: (495) 718-88-00

Ukraine: 03124, Kiev-49, b. I. Lipse,8, «Ergokom» ТПК PP Tel.: (+380-44) 454-11-98

Byelorus: 220033 Minsk, Fabrichnaya str. 26, r.18b «Fiatos» ТПЧУП Tel.: (+375-17) 298-12-60

Kazakhstan: 070010, Ust-Kamenogorsk, Karbysheva str. 24, «Kazmedimport» ТОО Tel.: (7232) 55-89-97

Uzbekistan: 700169, Tashkent, General Uzakov str.1, «ELD TIB Makhsulot» ООО. Tel.: (99871) 144-90-55

Singapore: LITTLE DOCTOR INTERNATIONAL (S) PTE. LTD., Yishun Central P.O. Box 9293 Singapore 917699. Fax: 65-62342197

CERTIFICATION AND THE MANUFACTURER INFORMATION

LD-N057 model corresponds the following EC-directive low voltage directive 73/23/EEC and directive 93/42/EEC. Product of Little Doctor International (S) Pte. Ltd. 35 Selegie Road #09-05 Parklane Shopping Centre, Singapore 188307 Fax: (65) 62342197. E-mail:ld@singaporemail.com

Exporter: LITTLE DOCTOR INTERNATIONAL (S) PTE. LTD. Manufacturer: SHANGHAI LITTLE DOCTOR ELECTRONIC CO., LTD., No. 4514-1 Caoan Road, Jiading District, Shanghai 201804, P.R. China Tel: +86 21 69596437 Fax: +86 21 69596439

WWW.LITTLEDOCTOR.SG

Information about LD brand medical appliances (specifications, operating and storage conditions, guarantee services etc.) can be obtained at the upper mentioned web-site.

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© Registered trademarks of Little Doctor International (S) Pte. Ltd.
© Little Doctor International (S) Pte. Ltd., 2006-2010